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
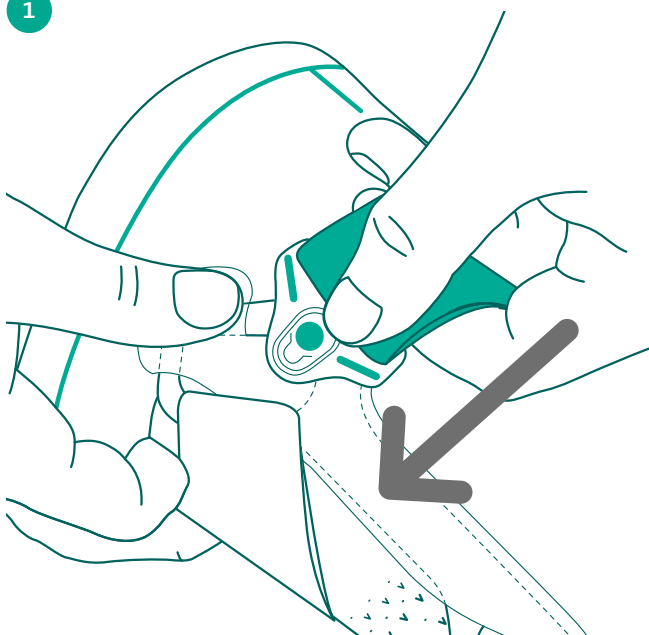
### EN Before application

Open the green and white marked quick fit buckles of the brace. Push the buckles forward until you can lift it out through the keyhole (picture 1 and 2).

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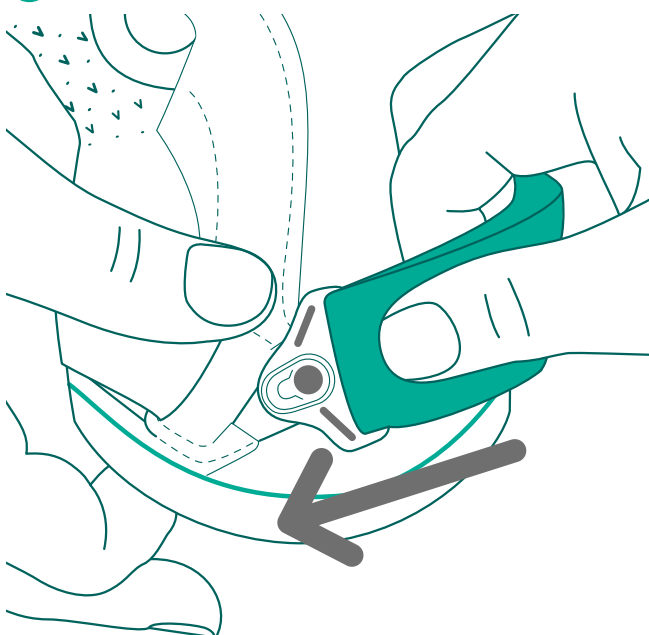
### EN Application

Pull the brace up until the joint centre is approximately at the middle of the kneecap (picture 3).

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

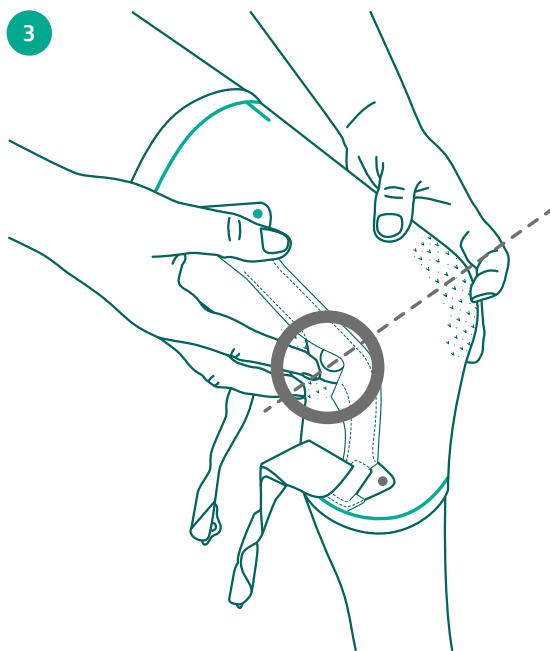
EN Make sure that the brace is straight and not twisted on the leg (picture 4).

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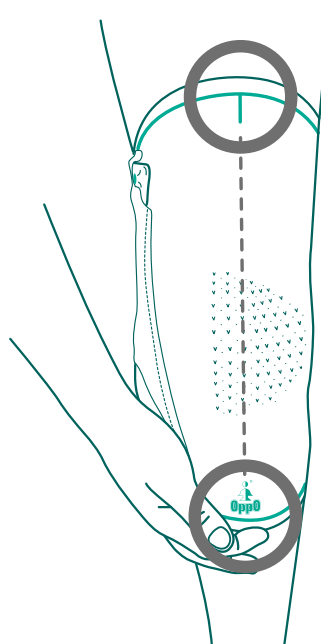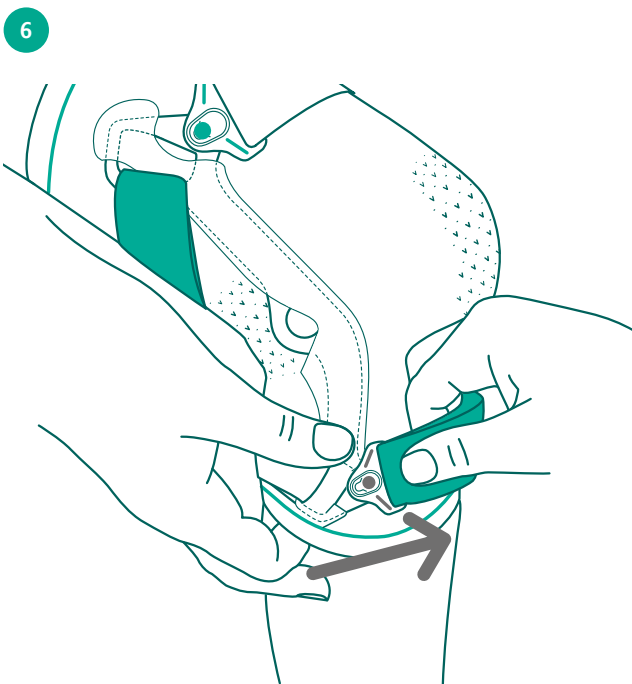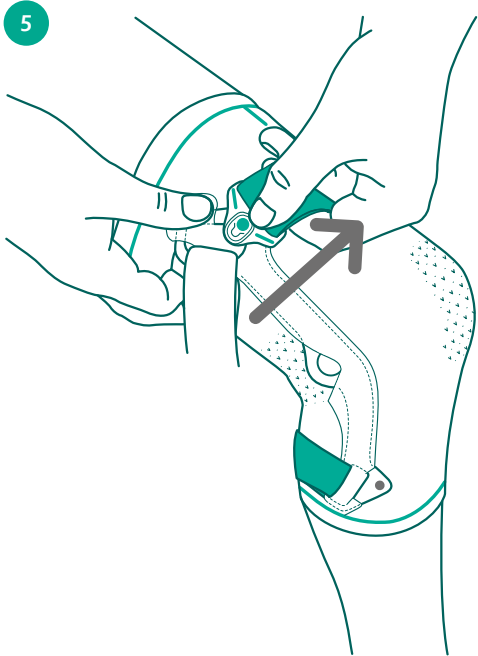
**EN** Now close the green marked quick fit buckle (picture 5). Now close the white marked quick fit buckle (picture 6).

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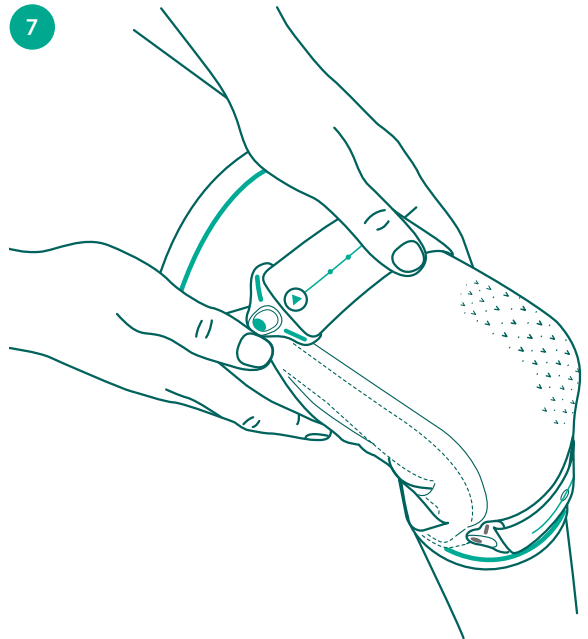
**EN** Your Orthotist has pre-set the brace and explained the orientation value to you. If you require more or less tension adjust the length of straps by using the velcro fasteners (picture 7).

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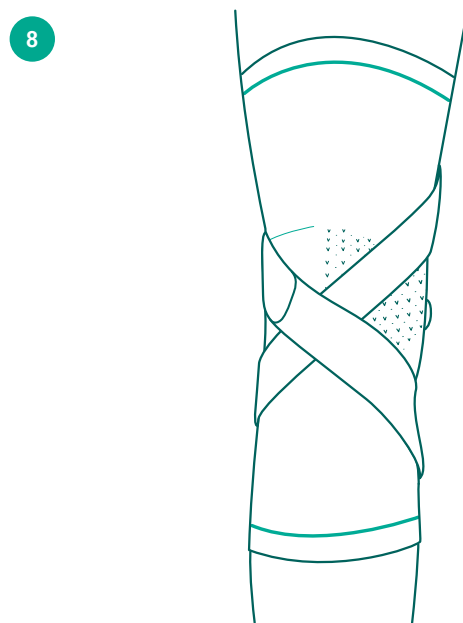**EN** Take a few steps to verify the correct position on the leg. If necessary, optimize the position again (picture 8).

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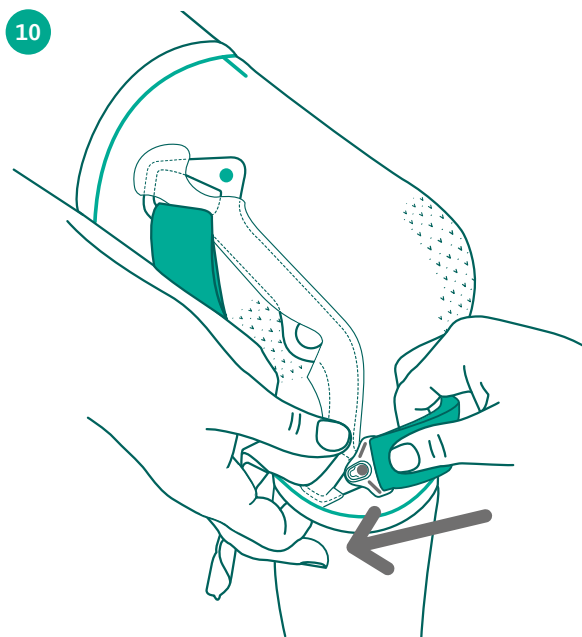
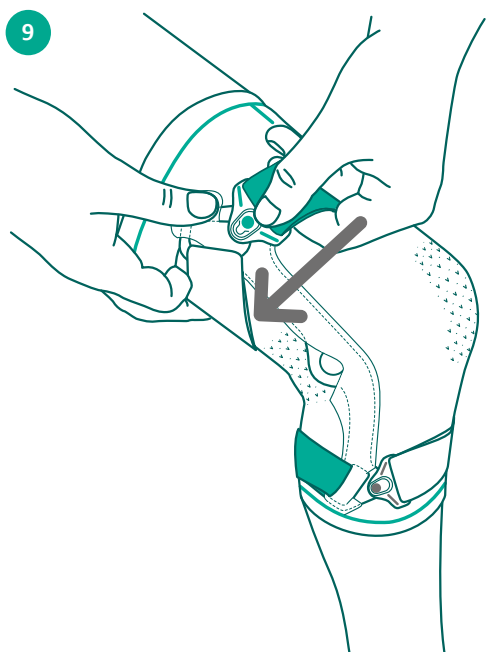
**EN Brace removal**

Bend the leg to about 30° and open the two green and white marked quick fit buckles (picture 9 and 10).

Pull the brace downwards. Close the quick fit buckles.

# INSTRUCTION FOR USE OPPO GENU X

**Dear Customer,**  
thank you for choosing an OPPO medical product. Please read through these instructions for use carefully. If you have any questions, please contact your doctor or medical professional.

This instruction for use is also available as a PDF-Documents and can be downloaded from [www.oppomedical.de](http://www.oppomedical.de). Visually impaired and blind people can also have the PDF-Document read out to them using suitable software.

## 1. INTENDED PURPOSE

The OPPO GENU X is a medical device. It is a brace for pain relief from osteoarthritis and degenerative meniscal tears through unicompartamental unloading of the knee.

## 2. INTENDED USER

This device is intended for people with unicompartamental knee joint osteoarthritis or meniscal tears.

## 3. INDICATIONS FOR USE

- › Mild to moderate medial osteoarthritis
- › Degenerative meniscal tears
- › Cartilage bone lesions
- › Bone bruise
- › Other knee conditions that may benefit from medial unloading
- › Condition after cartilage surgery or cartilage build-up with necessary medial postoperative unloading

## 4. CLINICAL BENEFIT

The brace provides a targeted relief and stabilization for more movement with osteoarthritis of the knee.

## 5. CONTRAINDICATIONS

Bilateral osteoarthritis (medial and lateral).

## 6. BODY SEGMENT

Knee joint.

## 7. PERFORMANCE CHARACTERISTICS

The brace adopts a series of advanced product designs. The ergonomic hinged stays working with the straps build up a 3-point system to provide dynamic support. This system generates opposite forces on target joint to ease excessive load on unicompartamental of the knee. The force adjusted dynamically accompany with the range of motion to allow more pain-free movement.

## 8. WARNING / PRECAUTION

- › The product provides its unloading effect during physical activity.
- › For first time use, we recommend that you gradually increase the usage time while building up tolerance to the brace.
- › Remove your brace during prolonged periods of rest.
- › Use the brace only as indicated and in accordance with the further instructions of the medical professional.
- › If used with other products, consult your medical professional.
- › Do not make any accidental changes to the product, which may alter the effect and cause health damage. Warranty and liability are excluded in such cases.
- › Avoid contact with fatty and acidic products, ointments or lotions.
- › Outside applied braces may cause local pressure symptoms or, in rare cases, constrict passing blood vessels or nerves.
- › If you notice changes or increasing discomfort while wearing the product, stop further use and contact your medical professional.
- › Regular practitioner supervision is recommended for users with:
  - Skin diseases/injuries in the treated part of the body, especially in cases of inflammation, scars with swelling, redness and overheating
  - Varicose veins (varicosis)
  - Strong Baker Cyst
- › Do not use this device if there is any damage on attached accessories, components, or the device itself.

## 9. NOTES ON REUSE

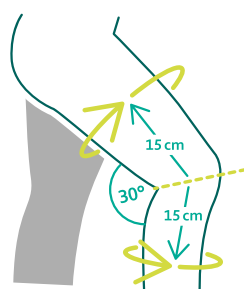
This product is intended and tested for use on a single patient. Use on more than one patient is excluded.

## 10. ASSEMBLY INSTRUCTIONS

The product must be fitted by an Orthotist or qualified medical professional.

## 11. ASSEMBLY INSTRUCTIONS

### Measurement Guide

Measure the circumference of the thigh at 15 cm (6 inches) above the mid-patella and measure the circumference of the calf at 15 cm (6 inches) below the mid-patella with knee fully extended.

- › See page 6 to 10 for step-by-step instructions on how to put the orthosis on.

**Hersteller Manufacturer:**

OPPO MEDICAL INC.  
Seattle City Center  
1420 Fith Ave., Ste. 220080  
Seattle WA 98101, USA  
info@oppomedical.com  
www.oppomedical.com  
www.oppomedical.de

**EU Bevollmächtigter  
EU Authorized representative:**

MT Promedt Consulting GmbH  
Altenhofstr. 80  
66386 St. Ingbert, Germany  
info@mt-procons.com  
www.mt-procons.com

**EU Importeur EU Importer:**

veomedical GmbH  
hallo@veomedical.de  
www.veomedical.de

**Artikelnummer Article Number:**  
324D100155U0

**Datum Date:**  
09.2023, 1.1 R Version